



## Proper Care of Aqua Creek Pool & Spa Lifts

PROCEDURE	BEFORE EACH USE	DAILY	WEEKLY	MONTHLY
Check battery level indicator on control box	✓			
Run the lift through a complete test cycle (fully extend actuator) to verify it is functioning properly	✓			
Wash lift w/ fresh water & dry w/ clean, non-abrasive cloth		✓		
Recharge battery (for frequent use, we recommend a second battery)		✓		®
Visually inspect lift for damage, corrosion, & loose or missing hardware		✓		
Check all contact points for damage and/or corrosion			✓	
Repair, clean, & apply dielectric grease to all battery contact points			✓	
Thoroughly clean lift frame and apply a liberal coat of car wax to maintain the lift's finish				✓
Check all Warning and Cautionary labels to make sure they are not faded or worn. Replace as needed.				✓





### PROPER CARE NOTES

- ***Use only fresh water to wash your lift. Do not wash with pool water***
- Use only clean, soft, non-abrasive cloths to wipe down your lift
- Do not store lift in pump room or near chemicals
- Do not allow children to play on or around this lift
- When cleaning the lift do not spray water directly on control box or battery <sup>®</sup>
- The lift may need to be bonded according to local code requirements

### STAIN REMOVAL TIPS:

*To remove stains, wash affected area with mild dish soap. For stubborn spots and stains try using soft scrub with some brisk rubbing. Use white epoxy paint to touch up. Apply wax to finish.*

For service and/or replacement parts contact Aqua Creek Products  
Toll free: (888) 687-3552 International: +1-(406) 549-0769

Copyright © 2018 Aqua Creek Products All Rights Reserved

