


**USE THIS MANUAL FOR PART:**  
F-UAPPS

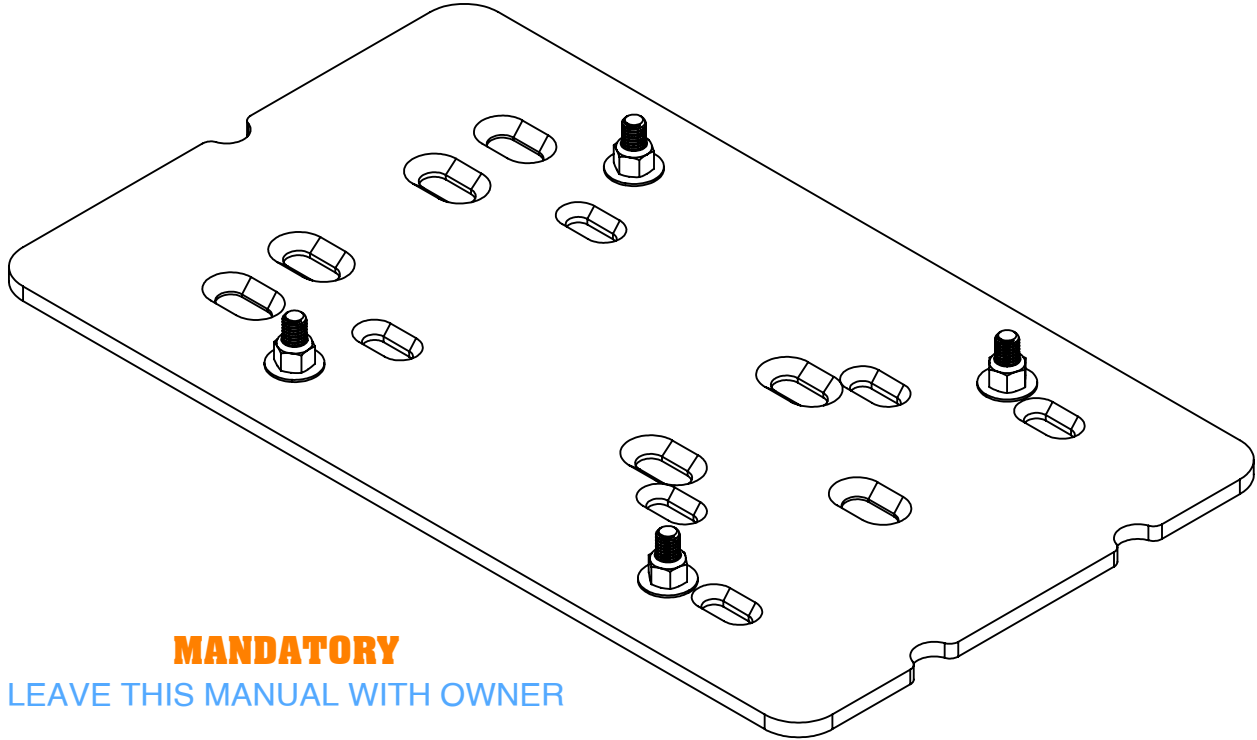

**MANDATORY**

LEAVE THIS MANUAL WITH OWNER

Read and follow all instructions.  
Lift safety can only be ensured if the lift  
is installed and operated according to  
these instructions.

**READ CAREFULLY**

Check entire box and all packing materials for parts. Before beginning assembly, read the instructions and identify parts using the figures and parts listed in this document.

It is critical that all parts be carefully inspected prior to installation. If any damage occurred in transit, Aqua Creek Products, LLC must be notified within three days of receipt of unit.

Proper installation cannot be overstressed, as an improper installation voids Aqua Creek's warranty and may affect the safety of the user.


• **FOR SAFETY REASONS**

- **NEVER** swim alone
- **NEVER** use the lift with a dry pool
- **NEVER** apply direct water pressure to the electronics
- **NEVER** permit children to play on or around the lift
- **DO NOT** allow children to use the lift without adult supervision

**Aqua Creek Products, LLC**

9889 Garrymore Lane  
Missoula, MT 59808  
Toll Free: (888) 687-3552  
Local/Intnl: (406) 549-0769  
[www.aquacreek.com](http://www.aquacreek.com)

**FIGURE A**

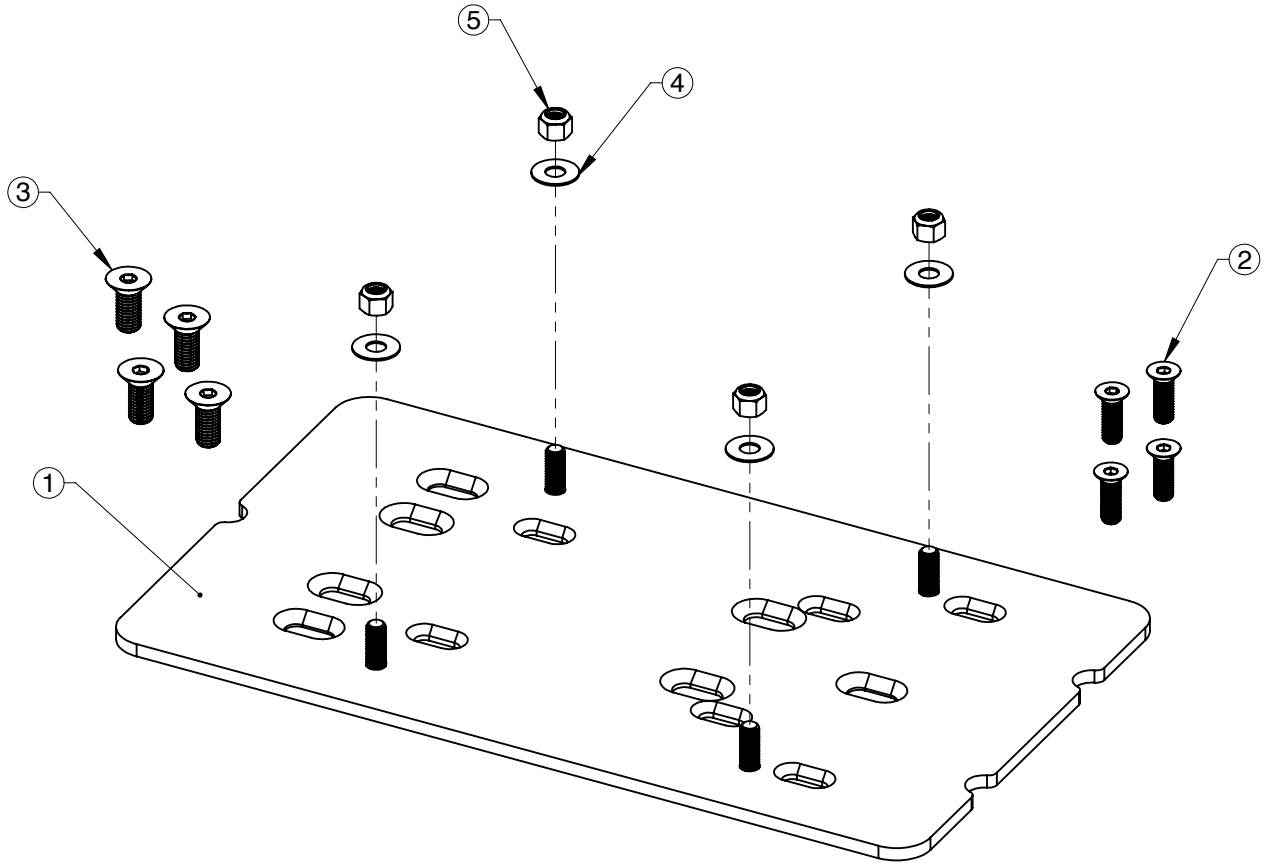

**ANCHOR - UNIVERSAL ADAPTER PLATE - INV03427**

ITEM	PART NO.	QTY.	REV.	DESCRIPTION
1	INV03426	1	P1	WELDMENT UNIVERSAL ADAPTER PLATE (RGR2, ADM)
2	INV00357	4	P1	BC - 1/2"-13X1 1/2" FLAT HEAD COUNTERSINK
3	INV03440	4	P1	BC - 5/8"-11X1 1/2"
4	INV01169	4	P1	WF - 1/2" FLAT WASHER
5	INV00733	4	P1	NN - 1/2"-13 NYLOCK NUT
6	INV00249	1	P1	SILICONE - BLISTER PACKET - 6 GRAMS

\*BLISTER PACKET NOT SHOWN





For service and/or replacement parts, contact Aqua Creek Products, LLC.  
 Toll Free: (888) 687-3552 Local/Intnl: (406) 549-0769 [www.AquaCreek.com](http://www.AquaCreek.com)

# BOLT PATTERN CONFIGURATIONS

## BOLT PATTERN OPTIONS FOR UNIVERSAL ADAPTER PLATE

△ NOTE PLATE ORIENTATION △

**FIGURE B**

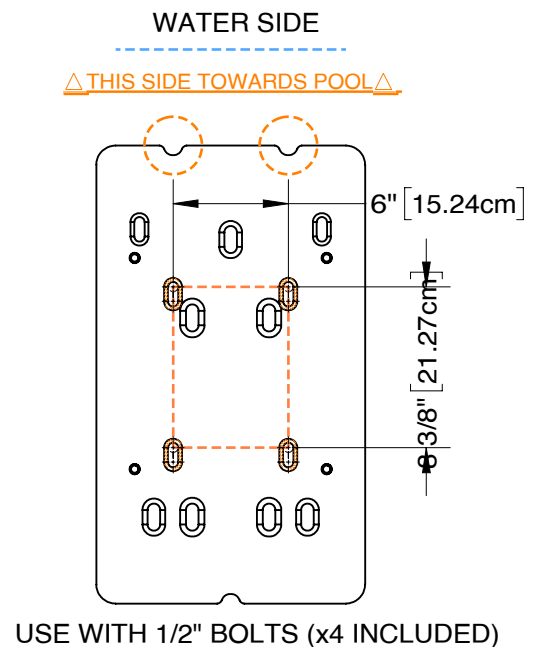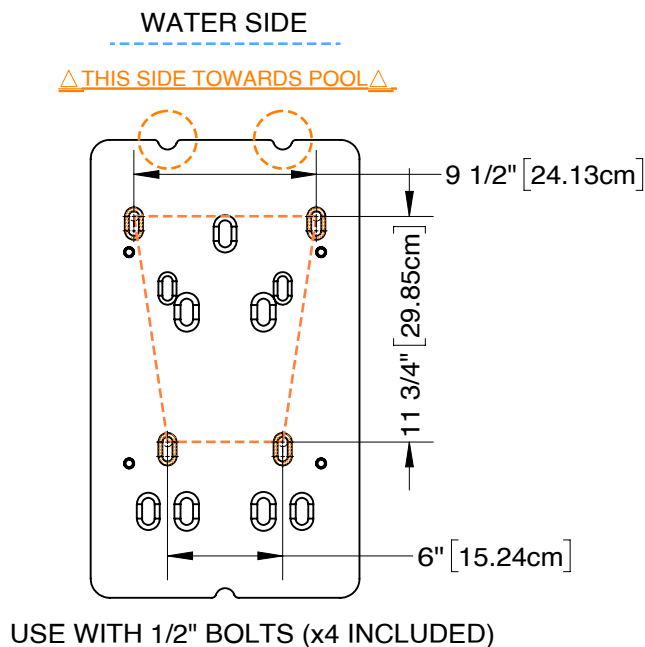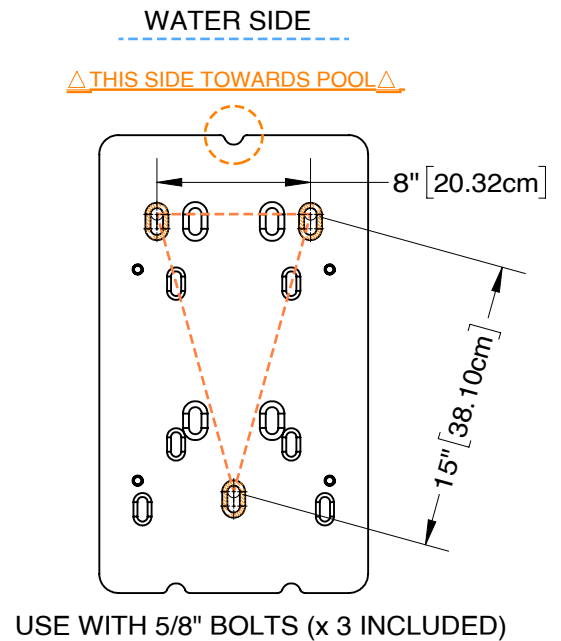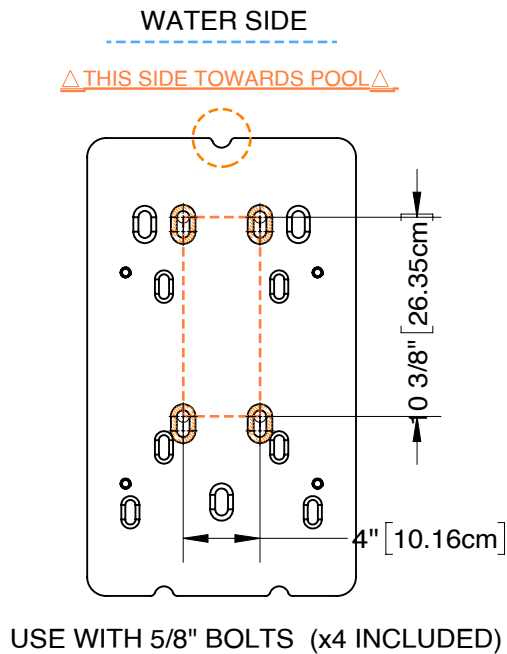

For service and/or replacement parts, contact Aqua Creek Products, LLC.  
Toll Free: (888) 687-3552 Local/Intnl: (406) 549-0769 [www.AquaCreek.com](http://www.AquaCreek.com)



**Aqua Creek Products**

Leaders in Recreation, Fitness and Ability

# PRODUCT INSTALLATION

## **STEP 1: LOCATE PLATE ORIENTATION**


The Universal Adapter Plate bolts directly to the pre-installed anchor bolt patterns. Locate the corresponding hole slots from the diagram on the previous page. Install the plate and bolt down until tightened.

\*NOTE THE PLATE ORIENTATION BEFORE LIFT INSTALLATION (see **FIGURE B** on the previous page).

## **STEP 2: INSTALL THE LIFT TO THE ADAPTER.**

With the Universal Adapter Plate bolted to the concrete, the lift is ready to be installed onto the plate. Use the included packet of anti-seize lube on the threaded studs before installing the nylock nuts. This will reduce corrosion and prevent stainless steel from galling. \*See **FIGURE C** below

**FIGURE C**

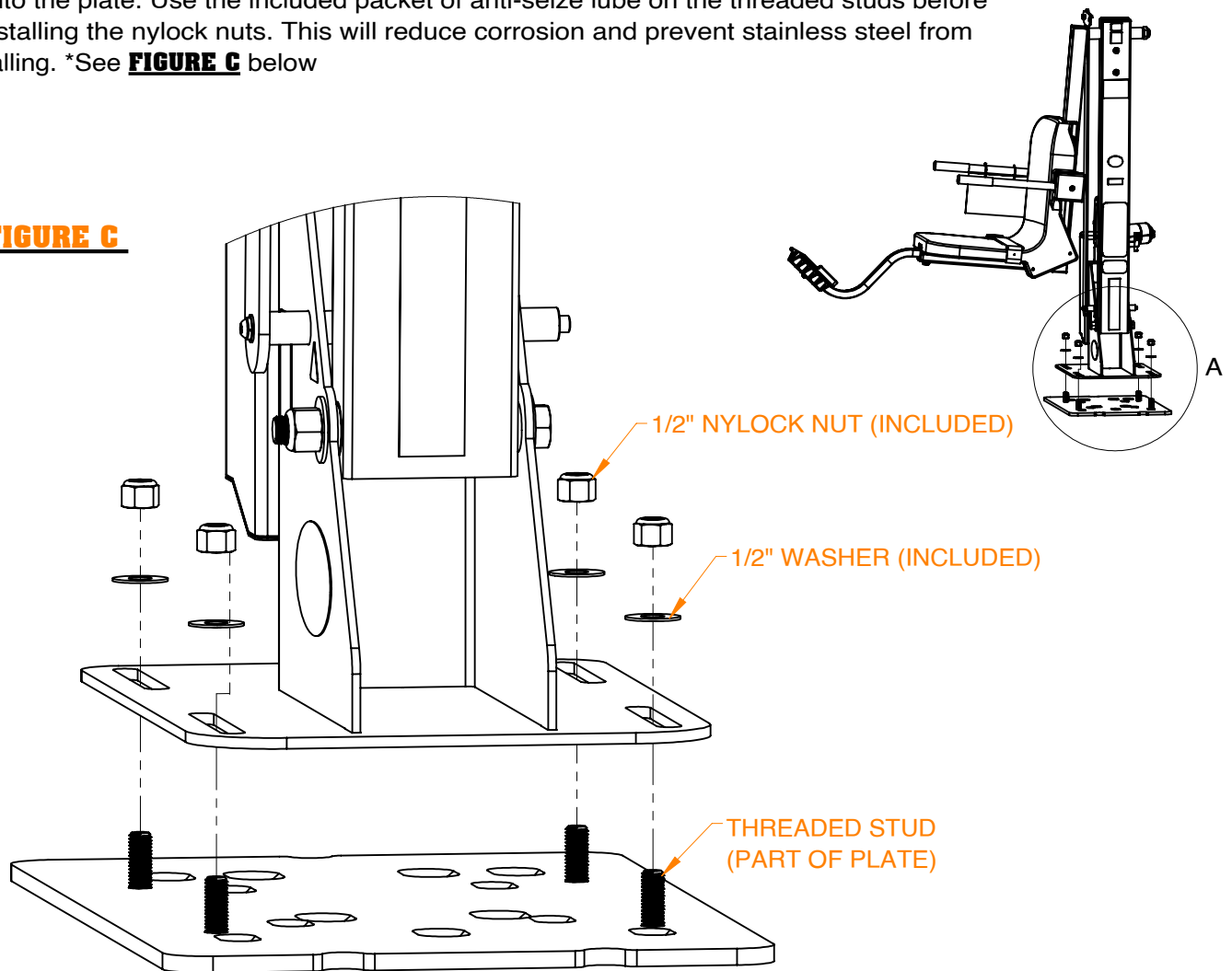

# PROPER PLACEMENT OF LIFT

For ADA applications, the front edge of the lift's base plate must be 13.00" from the pool or spa wall in order to meet the ADA requirement of 16.00" from the pool wall to the centerline of the chair. For non ADA applications the lift may be closer to the pool's edge. See FIGURE 4 below.

**FIGURE 4**

